

INCIDENTS | Add Patient

Note: All patient information can be added at any time, before or after patient creation.

01 Apply and Scan Wristband

- Scan wristband to add a patient
- Adding patients by tapping **NEW** increases the risk of a duplicate

02 Set Triage Condition (Color)

03 Add Patient Demographics

- Scan driver's license (DL), or enter manually
- Rapid: **Unknown** name, estimated **Age**, **Gender**

04 Take Photos (Optional)





- Face, injury, DL, belongings, family member, etc.

05 Chief Complaint (Optional)

- Type or use voice-to-text to describe major in-jury, vitals, intervention (Please begin the entry with "Country of Residence:" followed by the country and then the chief complaint.)

06 Create Patient

- If you **Select Destination** prior to hitting **Create**, you can immediately alert the receiving facility

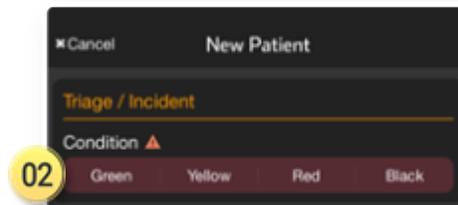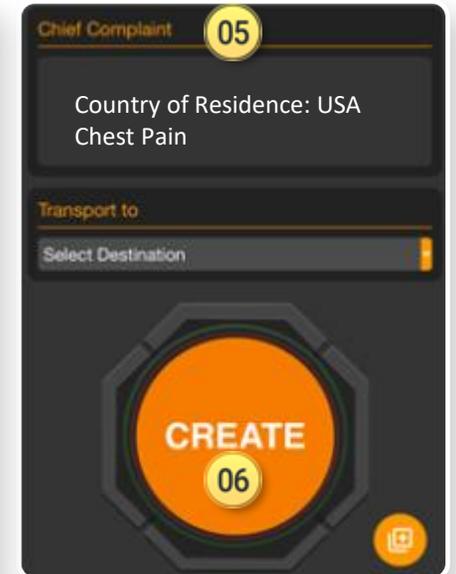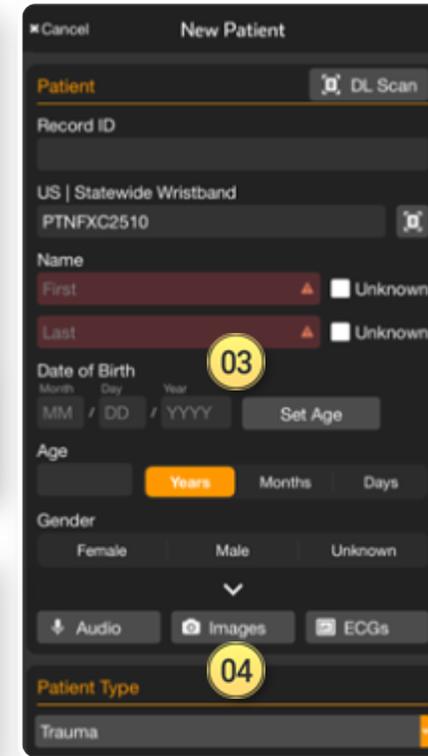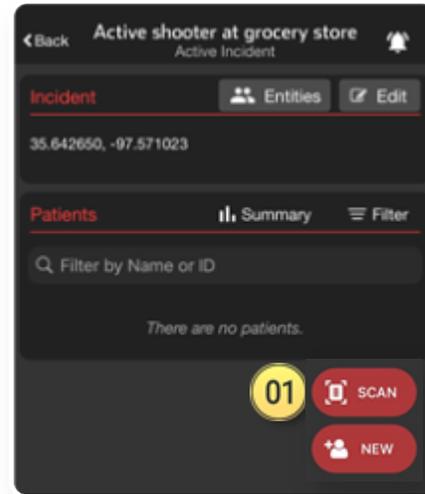

Quick Add

- 01 Scan Wristband
- 02 Triage Color
- 03 Unknown
- 06 Create